

FS907 DETACHABLE WHEELCHAIR



FS907 DETACHABLE WHEELCHAIR

PRODUCT OVERVIEW



FS907 Detachable Wheelchair



- Multi-function wheelchair with detachable foot rests and arm rests for easier patient transfer.
- Breathable woven nylon seat
- Lightweight aluminium frame

For illustration purposes only

DETACHABLE WHEELCHAIR

PRODUCT FEATURES



FOLDABLE BACKREST

Press on the lever to release and fold back rest downwards for both sides



FLIP UP ARM REST

Press button to release lock and flip up arm rest



DETACHABLE FOOTREST

Pull the lever to release lock and flip foot rest



STATIONARY HAND BRAKE

Push up to unlock and down to lock

For illustration purposes only

CARE INSTRUCTIONS

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Wheelchair.
- When getting in and out of the Wheelchair, please do not stand on the footplates.
- The footplates should be folded up / detached beforehand.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- Do not exceed the maximum safe working load.

FS907 DETACHABLE WHEELCHAIR

PRODUCT SPECIFICATIONS

Overall Length	95.5 cm or 37.5"
Overall Width (Wheel to Wheel)	58.5 cm or 23"
Overall Height	91.5 cm or 36"
Folded Length	75 cm or 29.5"
Folded Width	35 cm or 14"
Folded Height	71 cm or 28"
Seat Depth	41 cm or 16"
Seat Width	44.5 cm or 17.5"
Seat Height	51 cm or 20"
Arm Rest Height (Floor to Arm Rest)	71 cm or 28"
Back Rest Height	44.5 cm or 17.5"
Front Wheel Diameter	15.3 cm or 6"
Back Wheel Diameter	40.6 cm or 16"
Safe Working Load	100 kg
Overall Weight	14 kg

Check our our full range of detachable wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

