

WHEELCHAIRS | USER GUIDE

rainbow care

# ECL X260 ECLIPS RECLINING WHEELCHAIR



*For illustration purposes only*

**VERMEIREN**

## ECL X260 ECLIPS RECLINING WHEELCHAIR

### PRODUCT OVERVIEW




### ECL X260 Eclips Reclining Wheelchair

- Multi-function wheelchair with detachable foot rests and arm rests for easier patient transfer.
- Sized for children with a higher back rest and a seat block to prevent the child from sliding off the wheelchair while providing a good support.
- Breathable woven nylon seat
- Lightweight aluminium frame

## VERMEIREN RECLINING WHEELCHAIR

### PRODUCT FEATURES

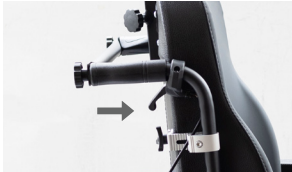

#### RECLINE BACKREST

Press on the lever to recline the back



#### FLIP UP ARM REST

Press button to release lock and flip up arm rest



#### DETACHABLE FOOTREST

Pull the lever to release lock to remove



#### ADJUST THE FOOTREST ANGLE

Push the lever to unlock or lock the footrest at the desired angle

*For illustration purposes only*

### CARE INSTRUCTIONS

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Wheelchair.
- When getting in and out of the Wheelchair, please do not stand on the footplates.
- The footplates should be folded up / detached beforehand.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- Do not exceed the maximum safe working load.

## ECL X260 ECLIPS RECLINING WHEELCHAIR

### PRODUCT SPECIFICATIONS

Overall Length	110 cm or 43.3"
Overall Width (Wheel to Wheel)	70 cm or 27.5"
Overall Height	130 cm or 51.2" (With headrest) 104 cm or 40.9" (Without headrest)
Seat Depth	44 cm or 17.3"
Seat Width	47 cm or 18.5"
Seat Height	56 cm or 22"
Reclining Angle	Up to 160°
Arm Rest Height (Floor to Arm Rest)	69 cm or 27.2"
Back Rest Height	77 cm - 95 cm or 30.3" - 37.4" (With headrest) 57 cm or 22.4" (Without headrest)
Front Wheel Diameter	15.2 cm or 6"
Back Wheel Diameter	61 cm or 24"
Safe Working Load	120 kg
Overall Weight	28.5 kg

Check our our full range of reclining wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

