

# FS903 DETACHABLE ELEVATING WHEELCHAIR



## FS903 DETACHABLE ELEVATING WHEELCHAIR

### PRODUCT OVERVIEW



### FS903 Detachable Elevating Wheelchair

- Multi-function wheelchair with detachable foot rests and flip up arm rests for easier patient transfer.
- Breathable woven nylon seat
- Lightweight aluminium frame

# ELEVATING WHEELCHAIR

## PRODUCT FEATURES



### FOLDABLE BACKREST

Press on the lever to release and fold back rest downwards for both sides



### FLIP UP ARM REST

Press button to release lock and flip up arm rest



### ELEVATING FOOTREST

Pull the lever to release elevating leg rest



### STATIONARY HAND BRAKE

Push up to unlock and down to lock

*For illustration purposes only*

## CARE INSTRUCTIONS

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Wheelchair.
- When getting in and out of the Wheelchair, please do not stand on the footplates.
- The footplates should be folded up / detached beforehand.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- Do not exceed the maximum safe working load.

## FS903 DETACHABLE ELEVATING WHEELCHAIR

### PRODUCT SPECIFICATIONS

Overall Length	104 cm or 40.9"
Overall Width (Wheel to Wheel)	63 cm or 24.8"
Overall Height	93 cm or 36.6"
Folded Length	72 cm or 28.3"
Folded Width	35 cm or 13.7"
Folded Height	67 cm or 26.4"
Seat Depth	41 cm or 16.1"
Seat Height	45 cm or 17.7"
Arm Rest Height (Floor to Arm Rest)	68 cm or 26.8"
Back Rest Height	46 cm or 18.1"
Front Wheel Diameter	20 cm or 7.9"
Back Wheel Diameter	58 cm or 22.8"
Safe Working Load	95 kg
Overall Weight	15 kg

Check our our full range of elevating wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

