

KW-S RECLINING GERIATRIC CHAIRS



For illustration purposes only

KW-S RECLINING GERIATRIC CHAIR

PRODUCT OVERVIEW




For illustration purposes only

Rainbow Care's Geriatric Chairs are equipped with large padded cushion for added comfort. It comes with a detachable table for convenience and multi-functionality. Built to recline from 90° to 130°, pull the knob to recline.

Designed according to ergonomic principles for users' comfort, this Reclining Geriatric Chair is padded with Cushion Upholstery in its Back Rest, Arm Pads and Seat. It can be conveniently adjusted to meet individuals' needs. Its Versatility puts users and caretakers mind at ease.

KW-S RECLINING GERIATRIC CHAIR

PRODUCT FEATURES

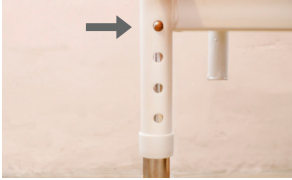

HEIGHT ADJUSTMENT

Press the catch to adjust the height of the chair



RECLINING LEVER

Pull the lever to recline the chair's back & foot rest



DETACHABLE TABLE

Insert table into the two holes underneath the arm rest then tighten the two knobs to secure it

For illustration purposes only

CARE INSTRUCTIONS

- When using Reclining Geriatric Chair, please pay attention to people and/or items around to avoid collision
- Ask for help if users have difficulty reclining the Geriatric Chair
- Use the safety belt to prevent user from sliding out of the chair
- Secure the table and tighten the two knobs underneath the arm rest before using
- When adjusting the height, ensure all 4 legs are leveled to the same height
- Ensure the Geriatric Chair is resting on an even surface
- Avoid uneven and wet surfaces
- Do not exceed the maximum load

PRODUCT SPECIFICATIONS

KW-S RECLINING GERIATRIC CHAIR

Overall Length	85 cm or 33"
Overall Width	69.5 cm or 27"
Overall Height	106 cm - 112 cm or 42" - 44"
Seat Depth	54 cm or 21"
Seat Width	48 cm or 19"
Seat Height	40 cm - 46 cm or 16" - 18"
Table Overall Length	65 cm or 25.5"
Table Overall Width	34 cm or 13"
Table Overall Thickness	1.5 cm or 0.6"
Materials	Body: Steel Seat: PVC Leather
Safe Working Load	100 kg
Overall Weight	20 kg

Check out our full range of Reclining Geriatric Chairs. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

