KW-W RECLINING GERIATRIC CHAIR





KW-W RECLINING GERIATRIC CHAIR

PRODUCT OVERVIEW



Rainbow Care's Geriatric Chairs are equipped with large padded cushion for added comfort. It comes with a detachable table for convenience and multi-functionality. Built to recline form 90° to 130°, pull the knob to recline.

Designed according to ergonomic principles for users' comfort, this Reclining Geriatric Chair is padded with Cushion Upholstery in its Back Rest, Arm Pads and Seat. It can be conveniently adjusted to meet individuals' needs. Its Versatility puts users and caretakers mind at ease.



KW-W RECLINING GERIATRIC CHAIR

PRODUCT FEATURES





HEIGHT ADJUSTMENT

Press the catch to adjust the height of the chair





RECLINING LEVER

Pull the lever to recline the chair's back & foot rest





LOCKABLE WHEELS

Push up to unlock and down to lock

For illustration purposes only

CARE INSTRUCTIONS

- When using Reclining Geriatric Chair, please pay attention to people and/or items around to avoid collision
- Ensure the wheels are locked before getting in and out of the Geriatric Chair.
- Ask for help if users have difficulty reclining the Geriatric Chair
- Use the safety belt to prevent user from sliding out of the chair
- · Pay attention to the shift of the Center of Gravity when on elevated grounds
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or damage the Geriatric Chair
- · Do not exceed the maximum load



PRODUCT SPECIFICATIONS

KW-W RECLINING GERIATRIC CHAIR

Overall Length	85 cm or 33"
Overall Width	67 cm or 27"
Overall Height	124.5 cm -135 cm or 49.0" - 53.1"
Seat Depth	54 cm or 21.3"
Seat Width	48 cm or 18.9"
Seat Height	45 cm - 56.5 cm or 17.7" - 22.2"
Table Overall Length	64 cm or 25.2"
Table Overall Width	40 cm or 15.7"
Table Overall Thickness	1.5 cm or 0.6"
Materials	Body: Steel Seat: PVC Leather
Safe Working Load	120 kg
Overall Weight	20 kg

Check out our full range of Reclining Geriatric Chairs. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.



© 6223 0904 • 6931 7425 © 8501 1878 ≅ sales@rainbowcare.com.sg ⊕ rainbowcare.com.sg ☐ @rainbowcare ☐ rainbowcaresg № Blk 3007 Ubi Road 1 #02-410 Singapore 408701