

HOME FURNISHINGS | USER GUIDE

rainbow care

# KW-WF-AA RECLINING GERIATRIC CHAIR



*For illustration purposes only*

PRODUCT SPECIFICATIONS

**KW-WF-AA RECLINING GERIATRIC CHAIR**

|                         |                                  |
|-------------------------|----------------------------------|
| Overall Length          | 85 cm or 33"                     |
| Overall Width           | 64 cm or 25.2"                   |
| Overall Height          | 110 cm - 115 cm or 43.3" - 45.3" |
| Seat Depth              | 46 cm or 18.1"                   |
| Seat Width              | 48 cm or 18.9"                   |
| Seat Height             | 46 cm - 51 cm or 18.1" - 20.1"   |
| Wheel Size              | 10 cm or 3.9"                    |
| Table Overall Length    | 64 cm or 25.2"                   |
| Table Overall Width     | 40 cm or 15.7"                   |
| Table Overall Thickness | 1.5 cm or 0.6"                   |
| Materials               | Body: Steel<br>Seat: PVC Leather |
| Safe Working Load       | 120 kg                           |
| Overall Weight          | 20 kg                            |

Check out our full range of Reclining Geriatric Chairs. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.



## KW-WF-AA RECLINING GERIATRIC CHAIR

### PRODUCT OVERVIEW



*For illustration purposes only*

Rainbow Care's Reclining Geriatric Chairs are equipped with large padded cushion for added comfort. It comes with a detachable table for convenience and multi-functionality. Built to recline from 90° to 130°, pull the knob to recline.

Designed according to ergonomic principles for users' comfort, this Reclining Geriatric Chair is padded with Cushion Upholstery in its Back Rest, Arm Pads, Seat, and Foot Rest. It can be conveniently adjusted to meet individuals' needs. Its Versatility puts users and caretakers mind at ease.

## KW-WF-AA RECLINING GERIATRIC CHAIR

### PRODUCT FEATURES



#### HEIGHT ADJUSTMENT

Press the catch to adjust the height of the chair



#### ADJUSTABLE ARM REST

Press the catch to adjust the height of the arm rest



#### RECLINING LEVER

Pull the lever to recline the chair's back & foot rest



#### LOCKABLE WHEELS

Push up to unlock and down to lock

*For illustration purposes only*

### CARE INSTRUCTIONS

- Ensure the wheels are locked before getting in and out of the Geriatric Chair.
- Use the safety belt to prevent user from sliding out of the chair
- Pay attention to the shift of the Center of Gravity when on elevated grounds
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or damage the Geriatric Chair
- Do not exceed the maximum load