

# FS696L ALUMINIUM MOBILE COMMODE



*For illustration purposes only*

## MOBILE COMMODE

### PRODUCT OVERVIEW



*For illustration purposes only*



*For illustration purposes only*

### FS696L Aluminium

It is made of lightweight aluminium frame and functions as a 2 in 1 commode where it is used as a commode chair and a shower chair.

- Foldable
- Detachable Bucket
- Height Adjustable

## MOBILE COMMODE

### PRODUCT FEATURES



#### HEIGHT ADJUSTMENT

Press the button and pull / push to the level that you want



#### REMOVE BUCKET

Lift up the seat cover to remove the bucket



#### LOCKABLE WHEELS

Press down to lock and lift up to unlock

*For illustration purposes only*

### CARE INSTRUCTIONS

- When using mobility commode, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Commode.
- When getting in and out of the commode, please do not stand on the footplates.
- The footplates should be folded up beforehand, or removed.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or damage the Commode.
- Do not exceed the maximum load.

## PRODUCT SPECIFICATIONS

## FS696L ALUMINIUM

Overall Length 45 cm or 17.7"

Overall Width 55 cm or 21.7"

Overall Height 81 cm - 91 cm  
31.9" - 35.8"

Seat Depth 40 cm or 15.7"

Seat Width 39 cm or 15.4"

Seat Height 50 cm - 60 cm  
19.7" - 23.6"

Front Wheel Diameter 10.2 cm or 4"

Back Wheel Diameter 10.2 cm or 4"

Safe Working Load 75 kg

Overall Weight 6 kg

Check our our full range of mobile commodes. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

