

BATHROOM AIDS | USER GUIDE

rainbow care

# DETACHABLE MOBILE COMMUNE



*For illustration purposes only*

## DETACHABLE MOBILE COMMODE

### PRODUCT OVERVIEW

FS692



FS692L



- Flip Up Arm Rests
- Detachable Seats
- Detachable Buckets
- Detachable Foot Rests



As a 3-in-1 Detachable Mobile Commode, a brilliant multi-tasker: a Commode Chair, a Shower Chair, as well as an Indoor Wheelchair.

- Foam Seat Cushions
- Lightweight Frame
- Detachable Bucket
- Detachable Foot Rest
- Flip Up Arm Rest
- 4 Wheels Lock


## DETACHABLE MOBILE COMMODE

### PRODUCT FEATURES



#### LEG REST

Pull the catch release to remove detachable leg rest

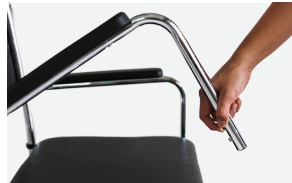

#### ARM REST

Press button to release lock and flip up arm rest



#### LOCKABLE WHEELS

Push up to unlock and down to lock

*For illustration purposes only*

### CARE INSTRUCTIONS

- When using mobility commode, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Commode.
- When getting in and out of the commode, please do not stand on the footplates.
- The footplates should be folded up beforehand, or removed.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or damage the Commode.
- Do not exceed the maximum load.

PRODUCT SPECIFICATIONS	FS692 CHROME	FS692L ALUMINIUM
Overall Length	87 cm or 34.3"	87 cm or 34.3"
Overall Width	52 cm or 20.5"	54 cm or 21.3"
Overall Height	93.5 cm or 36.8"	95 cm or 36.8"
Seat Depth	43 cm or 16.7"	43 cm or 16.7"
Seat Width	43 cm or 16.7"	43 cm or 16.7"
Seat Height (Without Cushion)	48.5 cm or	48.5 cm or
Seat Height (With Cushion)	53 cm or 20.9"	53 cm or 20.9"
Front Wheel Diameter	12.5 cm or 5"	12.5 cm or 5"
Back Wheel Diameter	12.5 cm or 5"	12.5 cm or 5"
Safe Working Load	100 kg	90 kg
Overall Weight	12 kg	10 kg

Check our our full range of mobile commodes. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

