

BATHROOM AIDS | USER GUIDE

rainbow care

# FS695S

## STAINLESS STEEL MOBILE COMMUNE



*For illustration purposes only*

# STAINLESS STEEL MOBILE COMMODE

## PRODUCT OVERVIEW



*For illustration purposes only*



*For illustration purposes only*

### FS695S Steel

The FS695S is made of a stainless steel lightweight frame and functions as a 3 in 1 commode where it is used as a commode chair, a shower chair and an indoor wheelchair.

- Flip Up Foot Rest
- Detachable Bucket
- Flip Up Arm Rest

## STAINLESS STEEL MOBILE COMMODE

### PRODUCT FEATURES



#### FOOT REST

Flip up to keep the foot rest



#### ARM REST

Pull the lever out to flip up the arm rest



#### LOCKABLE WHEELS

Push up to unlock and down to lock

*For illustration purposes only*

### CARE INSTRUCTIONS

- When using mobility commode, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Commode.
- When getting in and out of the commode, please do not stand on the footplates.
- The footplates should be folded up beforehand, or removed.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or damage the Commode.
- Do not exceed the maximum load.

---

PRODUCT SPECIFICATIONS	FS695S
Overall Length	73 cm or 28.7"
Overall Width	45 cm or 17.7"
Overall Height	97 cm or 38.2"
Seat Depth	43 cm or 16.9"
Seat Width	43 cm or 16.9"
Seat Height	45 cm or 17.7"
Front Wheel Diameter	12.5 cm or 5"
Back Wheel Diameter	12.5 cm or 5"
Safe Working Load	110 kg
Overall Weight	12 kg

---

Check our our full range of mobile commodes. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

