

BATHROOM AIDS | USER GUIDE

rainbow care

FT4211W STEEL MOBILE COMMUNE



For illustration purposes only

MOBILE COMMODE

PRODUCT OVERVIEW



For illustration purposes only



For illustration purposes only

FT4211W Steel

It is made of lightweight aluminium frame and functions as a 2 in 1 commode where it is used as a commode chair and a shower chair.

- Foldable
- Detachable Bucket
- Height Adjustable

MOBILE COMMODE

PRODUCT FEATURES



HEIGHT ADJUSTMENT

Press the button and pull / push to the level that you want



REMOVE BUCKET

Lift up the seat cover to remove the bucket



LOCKABLE WHEELS

Press down to lock and lift up to unlock

For illustration purposes only

CARE INSTRUCTIONS

- When using mobility commode, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Commode.
- When getting in and out of the commode, please do not stand on the footplates.
- The footplates should be folded up beforehand, or removed.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or damage the Commode.
- Do not exceed the maximum load.

PRODUCT SPECIFICATIONS FT4211W STEEL

| | |
|----------------------|-------------------------------------|
| Overall Length | 54 cm or 21.3" |
| Overall Width | 53 cm or 20.9" |
| Overall Height | 82.5 cm - 92.5 cm 32.5" or 36.4" |
| Seat Depth | 41.5 cm or 16.3" |
| Seat Width | 34.5 cm or 13.6" |
| Seat Height | 52.5 cm - 62.5 cm 20.7" - 24.6" |
| Front Wheel Diameter | 10.2 cm or 4" |
| Back Wheel Diameter | 10.2 cm or 4" |
| Safe Working Load | 80 kg |
| Overall Weight | 7.5 kg |

Check our our full range of mobile commodes. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

