

HOME FURNISHINGS | USER GUIDE

rainbow care

KW-S RECLINING GERIATRIC CHAIR



For illustration purposes only

KW-S RECLINING GERIATRIC CHAIR

PRODUCT OVERVIEW



For illustration purposes only

Rainbow Care's Geriatric Chairs are equipped with large padded cushion for added comfort. It comes with a detachable table for convenience and multi-functionality. Built to recline from 90° to 130°, pull the knob to recline.

Designed according to ergonomic principles for users' comfort, this Reclining Geriatric Chair is padded with Cushion Upholstery in its Back Rest, Arm Pads and Seat. It can be conveniently adjusted to meet individuals' needs. Its Versatility puts users and caretakers mind at ease.

KW-S RECLINING GERIATRIC CHAIR

PRODUCT FEATURES



HEIGHT ADJUSTMENT

Press the catch to adjust the height of the chair



ADJUSTABLE ARM REST

Press the catch to adjust the height of the arm rest



RECLINING LEVER

Pull the lever to recline the chair's back & foot rest



DETACHABLE TABLE

Insert table into the two holes underneath the arm rest then tighten the two knobs to secure it

For illustration purposes only

CARE INSTRUCTIONS

- Ensure the wheels are locked before getting in and out of the Geriatric Chair.
- Use the safety belt to prevent user from sliding out of the chair
- Pay attention to the shift of the Center of Gravity when on elevated grounds
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or damage the Geriatric Chair
- Do not exceed the maximum load

PRODUCT SPECIFICATIONS

KW-S RECLINING GERIATRIC CHAIR

Overall Length	85 cm or 33"
Overall Width	64 cm or 25.2"
Overall Height	110 cm - 115 cm or 43.3" - 45.3"
Seat Depth	46 cm or 18.1"
Seat Width	48 cm or 18.9"
Seat Height	46 cm - 51 cm or 18.1" - 20.1"
Table Overall Length	64 cm or 25.2"
Table Overall Width	40 cm or 15.7"
Table Overall Thickness	1.5 cm or 0.6"
Materials	Body: Steel Seat: PVC Leather
Safe Working Load	120 kg
Overall Weight	20 kg

Check out our full range of Reclining Geriatric Chairs. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

