

# HBL-4 DETACHABLE ELEVATING WHEELCHAIR



## HBL-4 DETACHABLE ELEVATING WHEELCHAIR

### PRODUCT OVERVIEW



### HBL-4 Detachable Elevating Wheelchair

- Multi-function wheelchair with flipping away the foot rests and the flip up arm rests for easier patient transfer.
- Breathable woven nylon seat
- Lightweight aluminium frame

## ELEVATING WHEELCHAIR

### PRODUCT FEATURES



#### FOLDABLE BACKREST

Press on the lever to release and fold back rest downwards for both sides



#### FLIP UP ARM REST

Squeeze lever to release lock and flip up arm rest



#### FLIP-AWAY FOOTREST

Pull the lever to release lock and flip the foot rest



#### STATIONARY HAND BRAKE

Push up to unlock and down to lock

*For illustration purposes only*

### CARE INSTRUCTIONS

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Wheelchair.
- When getting in and out of the Wheelchair, please do not stand on the footplates.
- The footplates should be folded up / detached beforehand.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- Do not exceed the maximum safe working load.

## HBL-4 DETACHABLE ELEVATING WHEELCHAIR

### PRODUCT SPECIFICATIONS

Overall Length	112 cm or 44.1"
Overall Width (Wheel to Wheel)	61 cm or 24.0"
Overall Height	88.5 cm or 34.8"
Folded Length	73 cm or 28.7"
Folded Width	32 cm or 12.6"
Folded Height	69 cm or 27.2"
Seat Depth	42 cm or 16.5"
Seat Height	48 cm or 18.9"
Arm Rest Height (Floor to Arm Rest)	68 cm or 26.8"
Back Rest Height	40 cm or 15.7"
Front Wheel Diameter	16 cm or 6.3"
Back Wheel Diameter	57 cm or 22.4"
Safe Working Load	100 kg
Overall Weight	14.5 kg

Check our our full range of elevating wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

