

WHEELCHAIRS | USER GUIDE

rainbow care

FS800-F5 TRAVEL WHEELCHAIR



For illustration purposes only

FS800-F5 TRAVEL WHEELCHAIR

PRODUCT OVERVIEW



Lightweight Wheelchair

- Easy to fold and operate
- Multi-function wheelchair with fold-up foot rests for easier patient transfer.
- Foldable back rest for easy storage
- Ventilated and cushioned seat upholstery
- Aluminium lightweight frame
- Pull-Trolley attachment for easy mobility

LIGHTWEIGHT WHEELCHAIR

PRODUCT FEATURES



FLIP UP ARM REST

Flip up arm rest by pushing up



HANDLE BRAKES

Push up handle lever fully to lock and press lower lever to unlock



FLIP UP FOOT REST

Flip up foot rest by pushing upwards

For illustration purposes only

CARE INSTRUCTIONS

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Wheelchair.
- When getting in and out of the Wheelchair, please do not stand on the footplates. The footplates should be folded up / detached beforehand.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- Do not exceed the maximum safe working load.

FS800-F5 TRAVEL WHEELCHAIR

PRODUCT SPECIFICATIONS

Overall Length	90 cm or 35.4"
Overall Width (Wheel to Wheel)	56.5 cm or 22.2"
Overall Height	93 cm or 36.6"
Folded Length	40 cm or 15.7"
Folded Width (Wheel to Wheel)	47 cm or 18.5"
Folded Height	80.5 cm or 31.7"
Seat Depth	37 cm or 14.6"
Seat Width	48 cm or 18.9"
Seat Height (Floor to Seat Base)	50 cm or 19.7"
Arm Rest Height (Floor to Arm Rest)	74 cm or 29.1"
Back Rest Height	33 cm or 13.0"
Front Wheel Diameter	15 cm or 5.9"
Back Wheel Diameter	28 cm or 11.0"
Safe Working Load	90 kg
Overall Weight	8 kg

Check our our full range of detachable wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

