

WHEELCHAIRS | USER GUIDE

rainbow care

# FS800 TRAVEL WHEELCHAIR



*For illustration purposes only*

## FS800 TRAVEL WHEELCHAIR

### PRODUCT OVERVIEW



#### Lightweight Wheelchair

- Easy to fold and operate
- Multi-function wheelchair with fold-up foot rests for easier patient transfer.
- Foldable back rest for easy storage
- Ventilated and cushioned seat upholstery
- Aluminium lightweight frame
- Pull-Trolley attachment for easy mobility

# LIGHTWEIGHT WHEELCHAIR

## PRODUCT FEATURES



### FLIP UP ARM REST

Flip up arm rest by pushing up



### HANDLE BRAKES

Push up handle lever fully to lock and pull red knob to unlock



### FLIP UP FOOT REST

Flip up foot rest by pushing upwards

*For illustration purposes only*

## CARE INSTRUCTIONS

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Wheelchair.
- When getting in and out of the Wheelchair, please do not stand on the footplates. The footplates should be folded up / detached beforehand.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- Do not exceed the maximum safe working load.

## FS800 TRAVEL WHEELCHAIR

### PRODUCT SPECIFICATIONS

|                                     |                 |
|-------------------------------------|-----------------|
| Overall Length                      | 84 cm or 33.1”  |
| Overall Width (Wheel to Wheel)      | 49 cm or 19.3”  |
| Overall Height                      | 89 cm or 35.0”  |
| Folded Length                       | 49 cm or 19.4”  |
| Folded Width (Wheel to Wheel)       | 17 cm or 6.7”   |
| Folded Height                       | 74 cm or 29.1”  |
| Seat Depth                          | 35 cm or 13.8”  |
| Seat Width                          | 39 cm or 15.4”  |
| Seat Height (Floor to Seat Base)    | 47 cm or 18.5”  |
| Arm Rest Height (Floor to Arm Rest) | 69 cm or 27.2”  |
| Back Rest Height                    | 33 cm or 13.0”  |
| Front Wheel Diameter                | 15.2 cm or 6.0” |
| Back Wheel Diameter                 | 20.3 cm or 8.0” |
| Safe Working Load                   | 80 kg           |
| Overall Weight                      | 7 kg            |

Check our our full range of detachable wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

