

FS908-21 DELUXE DETACHABLE WHEELCHAIR



FS908-21 DELUXE DETACHABLE WHEELCHAIR

PRODUCT OVERVIEW



FS908-21 Delux Detachable Wheelchair

- Multi-function wheelchair with detachable foot rests and arm rests for easier patient transfer.
- Breathable woven nylon seat
- Lightweight aluminium frame

DETACHABLE WHEELCHAIR

PRODUCT FEATURES



FOLDABLE BACKREST

Press on the lever to release and fold back rest downwards for both sides



FLIP UP ARM REST

Press button to release lock and flip up arm rest



DETACHABLE FOOT REST

Pull the lever to release lock and flip the foot rest



STATIONARY HAND BRAKE

Push up to unlock and down to lock

For illustration purposes only

CARE INSTRUCTIONS

- When using the Wheelchair, please pay attention to people and/or items around to avoid collision.
- Lock the wheels before getting in and out of the Wheelchair.
- When getting in and out of the Wheelchair, please do not stand on the footplates.
- The footplates should be folded up / detached beforehand.
- Pay attention to the shift of the Center of Gravity when on elevated grounds.
- When passing through doors or arches, make sure that there is enough room to avoid causing injuries to the user, or the Wheelchair.
- Do not exceed the maximum safe working load.

FS908-21 DELUXE DETACHABLE WHEELCHAIR

PRODUCT SPECIFICATIONS

Overall Length	105 cm or 41.3"
Overall Width (Wheel to Wheel)	61 cm or 24"
Overall Height	89 cm or 35"
Folded Length	80 cm or 31.5"
Folded Width	38 cm or 15"
Folded Height	68 cm or 26.8"
Seat Depth	39.5 cm or 15.6"
Seat Width	45.7 cm or 18"
Seat Height	47 cm or 18.5"
Arm Rest Height (Floor to Arm Rest)	67 cm or 26.4"
Back Rest Height	40.7 cm or 16"
Front Wheel Diameter	20.3 cm or 8"
Back Wheel Diameter	60 cm or 23.5"
Safe Working Load	100 kg
Overall Weight	14 kg

Check our our full range of detachable wheelchair. When making comparisons, our best suggestion would be for you to think about your environment and consider how certain task like transferring someone is done before deciding.

Scan the QR code to find out more about the product. To scan, use your camera app or a QR code reader on your device.

